

Parking brake installation

Have a question? Give us a call at 1-877-345-4748 or email us at support@gogrit.us.

Riders of our 20" seat width chair need to install their own parking brakes before riding their new Freedom Chair. It's a quick process and is easiest to perform before attaching the wheels.

1

The right and left brakes are labeled 'R' and 'L'. Slide them onto their corresponding sides (from the rider's perspective) of the chair. Be sure that when the brake lever is hanging down that the two brake bars are facing the front of the chair, as shown in the photo.



2

Carefully thread the screw into the chair from the bottom. Make sure it engages the threads, then use the provided allen key to tighten it in place. Make sure it is tight and all of the way in. That's it!

